

# Healthy Tracks™

## for Pets

### LANCING DEVICE INSTRUCTIONS FOR USE

For use only on Dogs and Cats. NOT FOR HUMAN USE.

For use with Healthy Tracks for Pets™ lancet to obtain a capillary whole blood sample for blood glucose testing.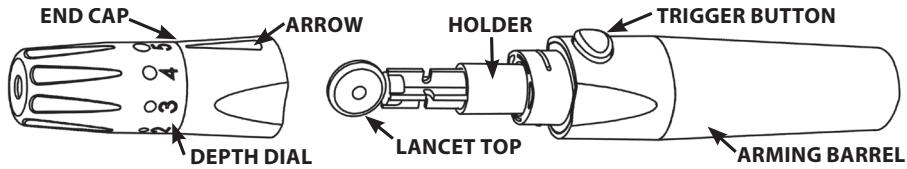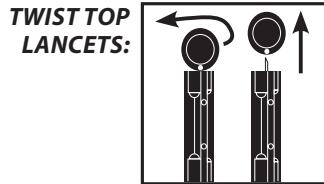

#### **WARNING!**


The lancing device is for single patient (dog or cat) use **ONLY. DO NOT** share it with other animals. **DO NOT** use on multiple animals. **ALL** parts of the lancing device are considered biohazardous and can potentially transmit infectious diseases, even after you have performed cleaning and disinfection.

#### **CAUTION!**

**Lancets are for single use only. Do not re-use. Do not use lancet if Lancet Top has been previously removed.** Wash your hands thoroughly with soap and water before and after handling the meter, lancing device, or test strips to prevent the transfer of blood-borne pathogens.

#### - R E A D Y -

- ① Pull off End Cap.  
Push lancet firmly into Holder.
- ② Remove Lancet Top.  
**Save Lancet Top for safe disposal of used lancet.**
- ③ Replace End Cap.

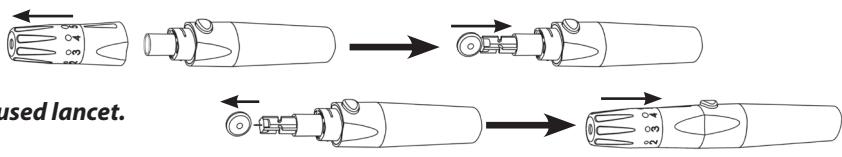
#### - S E T -

- ① Turn Depth Dial to desired lancing depth.  
**Finer adjustments can be made by setting Arrow between numbers.**

Suggested Lancing Depth:  
 • 3-5 – ear  
 • 4-5 – paw pad

- ② Hold End Cap in one hand and Arming Barrel in other. Gently pull Arming Barrel until you hear a click.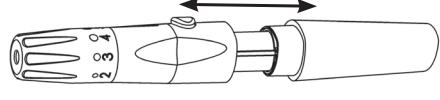


#### - L A N C E -

	<ol style="list-style-type: none"> <li>1. Wash your hands and the test site (e.g. ear or paw pad) with warm water and soap to ensure accurate results. Rinse and dry thoroughly. You may also use an alcohol wipe to clean the test site. Allow the test site to dry thoroughly before testing. Gently massage the test site or apply a warm cloth to help the blood drop form.</li> </ol>
	<ol style="list-style-type: none"> <li>2. Place lancing device tip firmly against the test site and press the Trigger Button. Remove lancing device and set aside. Allow blood drop to form. <b>Important!</b> When lancing the ear, it is recommended that you place a cotton ball on the opposite side of the animal's ear to protect your finger.</li> </ol>
<ol style="list-style-type: none"> <li>3. After testing, pull off End Cap. Place the saved Lancet Top flat on a firm surface and push the exposed lancet tip into the Lancet Top. Remove the lancet from the Holder and discard the used lancet into a suitable container.</li> </ol>	

A capillary blood sample can be taken from the ear or paw pad in dogs and cats.

**Ask your veterinarian for instructions on getting a blood sample from the ear or paw pad.**

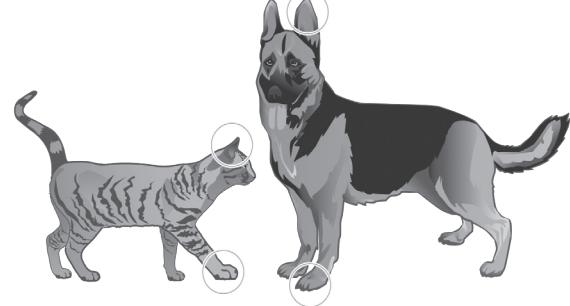Refer to the Owner's Booklet for complete information about the Test Buddy™ Pet-Monitoring Blood Glucose System and instructions on how to obtain a blood glucose reading.

#### - L A N C I N G D E V I C E C L E A N I N G A N D D I S I N F E C T I N G -

- Cleaning and disinfecting is a two-step process. The first step is cleaning. Cleaning removes any blood, liquids, dust, or dirt. The second step is disinfecting. Disinfecting removes most, but not all possible infectious agents, including blood-borne pathogens. **Cleaning must be done before disinfecting.**
- Clean the lancing device immediately after getting any blood on it or if it is dirty.
- Remove lancet from lancing device before cleaning.
- Clean and disinfect the lancing device at least once a week. The lancing device may be cleaned and disinfected once a day for a 3-year period.
- **Use only PDI Super Sani-Cloth® Wipes** (EPA Reg. No. 9480-4). Other disinfectants have not been tested and may damage the lancing device. PDI Super Sani-Cloth Wipes may be purchased at: Amazon.com, Officedepot.com (or your local Office Depot store), or Walmart.com.
- PDI Super Sani-Cloth Wipes have been tested on the lancing device for a total of 1,095 cleaning and disinfecting cycles, which is equal to cleaning and disinfecting the lancing device once a day for a 3-year period.

	<b>To Clean the Lancing Device:</b> <ol style="list-style-type: none"> <li>1. Wash hands thoroughly with soap and water.</li> <li>2. Thoroughly wipe the entire outside surface of the lancing device with one Super Sani-Cloth by wiping in circles 3 times. Discard the used cloth.</li> </ol>
	<b>To Disinfect the Lancing Device:</b> <ol style="list-style-type: none"> <li>1. <b>Always clean the lancing device immediately before disinfecting</b> (see instructions above).</li> <li>2. Remove a fresh Super Sani-Cloth and thoroughly wipe the entire outside surface of the lancing device by wiping in circles 3 times. Make sure that you go over the entire lancing device.</li> <li>3. Using fresh cloths, if needed, <b>make sure that all outside surfaces of the lancing device stay wet for 2 minutes by patting with the cloth.</b> Discard used cloths.</li> <li>4. Let the lancing device air dry thoroughly before using.</li> <li>5. Gently pull back the Arming Barrel and press the Trigger Button. You will hear a click if the lancing device is functioning properly.</li> <li>6. Wash hands thoroughly when finished.</li> </ol>

**Important!** Stop using the lancing device and contact Customer Care for assistance, if:

- Markings on lancing device are coming off
- Trigger Button is hard to push
- End Cap does not go back on
- Arming Barrel does not click when gently pulled back

Manufactured for:

**TRIVIDIA**  
HEALTH™  
Fort Lauderdale, FL 33309 USA  
1-800-803-6025

Made in China • S3HTK06 Rev. 2

Call Customer Care at 1-800-803-6025, Mon. - Fri., 8AM - 8PM EST.

© 09/2020 Trividia Health, Inc. Healthy Tracks for Pets, Test Buddy and the Trividia Health logo are trademarks of Trividia Health, Inc.

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